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EN
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY
NON - STERILE PRODUCT. STERILIZE DIRECTLY BEFORE USE

The tools are made of corrosive – resistant stainless steel.

HANDLING DENTAL TOOLS MADE OF STAINLESS STEEL

Special processes have been undertaken during the manufacture of the dental instruments to ensure corrosion-resistance. In spite of this, the durability of the dental instruments depends on their proper use and correct maintenance.

CLEANING, RINSING AND WASHING

The packaged instruments are non – sterile. Before sterilization, these new instruments should be washed thoroughly with warm water and using detergent. Before each use these dental instruments must be sterilized.

STERILIZATION

All (reusable) dental instruments must be disinfected, then washed out to remove detergent and then dried before sterilization. Ultrasonic cleansing is recommended in particular as the most effective sterilization. Instruments should be sterilized in accordance with national requirements.

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