PL

GLUCO-CHEX 2% gel

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GLUCO-CHEX 2% gel

Gel for root canals preparation

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: chlorhexidine digluconate 2%, base gel.

INTENDED USE

GLUCO-CHEX 2% GEL is intended for endodontic treatment tp facilitate mechanical root canal preparation. It lubricates the root canal during mechanical preparation. Lubricants in the product allow easy introduction of an instrument into the canal and minimize the risk of breaking the tool.

INSTRUCTIONS FOR USE

Before the treatment, immerse the attached applicator into ethyl alcohol and then place it onto the syringe. Place the product into the root canal with the canal instrument or with the attached applicator, while simultaneously preparing the canal mechanically. While preparing the canal, rinse it with physiological saline or distilled water.. A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use for patients who are hypersensitive to the product components.

CAUTION! Do not rinse with sodium hypochlorite as it may cause sedimentation of a brown deposit which is hard to remove.

PRECAUTIONS

Caution is advised when preparing the canal close to the apex area, as the tool may cause perforation of the canal. In case of contact, rinse immediately with plenty of water If swallowed, drink plenty of water. Do not induce vomiting. Seek medical advice. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Keep out of reach of children. Protect from sunlight. Expiry date is placed on the main container. Keep in original packaging below 25 °C.

Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe with 5 ml or 10 ml of the preparation, dispenser, connector, disposable applicators.

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