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EN	Root canal orifice indicator
<b>PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE TO BE USED BY DENTIST ONLY</b>	

COMPOSITION: methylene blue, propylene glycol, Ph. Eur. purified water.

INTENDED USE

Product CANAL DETECTOR is intended for use during endodontic treatment to identify the root canal orifices. Blue dye contained in the product penetrates into root canal orifices turning them blue and making it easier to find the main root canal orifices and location of any cracks.

INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator in alcohol (ethyl alcohol) and then fix it onto the syringe.

Remove the dentine where the root canal orifices are expected, insert 1-3 drops of preparation on the bottom of the tooth cavity, wait 5-10 seconds, then rinse and dry the tooth cavity. The points colored intense blue indicate the root canal orifices.

After using the product, withdraw the syringe piston to prevent overflow of preparation, screw off the applicator and then screw the white plug on the syringe.

Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients.

PRECAUTIONS

Use rubber dams while working with the product. Use disposable gloves, protective glasses and clothing. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging below 25°C. Keep out of reach of children. The expiry date is on the direct packaging. Once container is open, the expiry date doesn't change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Syringe with 2 ml of preparation, disposable brush applicators.

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