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EN Calcium hydroxide cement for filling cavities

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
TO BE USED BY DENTIST ONLY

COMPOSITION: calcium hydroxide, hydroxyapatite, calcium salts, anhydrous sodium carbonate, magnesium oxide.

INTENDED USE
HYDROCAL is used during dental treatment for temporary filling of the cavities and root canals and in direct and indirect pulp-capping treatment. The product remineralizes the hard tissues of tooth and is used as a temporary root canal or cavity dressing before the final filling. HYDROCAL is recommended for mixing pastes with selected liquids, usually with physiological saline.

INSTRUCTIONS FOR USE
Mix the HYDROCAL product with distilled water or physiological solution of salts until the desired consistency of paste is obtained. Fill the cavity or root canal. Leave the filling in the cavity or the root canal for 1 week to 30 days.

CONTRAINDICATIONS
Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS
Irritating to the mucous membranes of the oral cavity, skin and eyes. May cause serious eye damage.

In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting.

Drink plenty of water.

Seek medical advice.

Use a rubber-dam when working with the product.

Use protective gloves and protective clothing.

Any serious incident involving this product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

If inhaled: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Call a poison center or doctor if you feel unwell.

STORAGE
Store in original packaging below 25°C. Protect from moisture. Keep out of reach of children.

Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Container with 10 g of preparation.

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INTENDED USE
HYDROCAL is used during dental treatment for temporary filling of the cavities and root canals and in direct and indirect pulp-capping treatment.

The product remineralizes the hard tissues of tooth and is used as a temporary root canal or cavity dressing before the final filling. HYDROCAL is recommended for mixing pastes with selected liquids, usually with physiological saline.

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