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EN Light-curing rubber-dam liquid
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY

COMPOSITION: urethane resins, inorganic fillers

INTENDED USE
Light-curing rubber-dam liquid is intended for:

- gingival protection during dental treatments: whitening, etching, microabrasion, sandblasting etc.
- sealing rubber dams

Suitable for occlusal stamp technique.

INSTRUCTIONS FOR USE
Before the treatment immerse attached applicator in alcohol (ethyl alcohol) and then fix it onto the syringe.

For gingival protection during dental treatments:
Clean and dry the gingiva. With the use of appropriate instruments (eg. buccal and labial retractor) immobilize the lips and tongue in order to avoid contact with the liquid. Remove the syringe lock and install the included disposable applicator. Apply RUBBER-DAM LIQUID along the application area 4-6 mm wide and approximately 1.5-2 mm thick at the same time visually checking the amount of the applied preparation. If possible, squeeze RUBBER-DAM LIQUID through the interdental space. Cure the preparation for 20-30 seconds using a polymerisation lamp. Carry out the curing process evenly in a scanning motion. Proceed to the dental treatment. After the treatment is completed, using an instrument from one side, lift off and remove polymerised RUBBER-DAM LIQUID.

In order to seal a rubber dam:
Isolate the working area (tooth/teeth/alveolus) using RUBBER-DAM.
In order to seal the rubber dam, apply a layer of the liquid on the edge of the gingiva and the rubber dam. Cure the preparation for 20-30 seconds using a polymerisation lamp. Carry out the curing process evenly in a scanning motion. Proceed to the dental treatment. After the treatment is completed, using an instrument from one side, lift off and remove polymerised RUBBER-DAM LIQUID.

Immediately after use, withdraw the syringe plunger and place the original cap on the container to avoid polymerisation of the product in the syringe.
Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS
Do not use the product for patients with hypersensitivity to the product’s components.

PRECAUTIONS
Un-cured substance can cause irritation to the skin and eyes. Sustained contact with the skin can cause allergic reaction. In case of contact, immediately rinse with plenty of water. If it may cause respiratory irritation. Harmful when swallowed. If swallowed, do not induce vomiting. Rinse mouth with water. Contact a doctor.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which user or patient is domiciled.

STORAGE
Store in a tightly closed original container below 25°C. Protect against direct sunlight. Close tight after use. Keep out of reach of children.
The expiry date is on the product packaging. Once container is open, the expiry date doesn’t change if the container is securely closed after each use.

TRANSPORT
The manufacturer allows transport of the product for a period not longer than 10 days at a temperature not exceeding 40°C. Short-term transport within the given temperature range does not affect the safety and efficacy of the product throughout its shelf life.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Syringe containing 1,2 ml of the preparation.

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PRECAUTIONS
Un-cured substance can cause irritation to the skin and eyes. Sustained contact with the skin can cause allergic reaction. In case of contact, immediately rinse with plenty of water. If it may cause respiratory iration. Harmful when swallowed. If swallowed, do not induce vomiting. Rinse mouth with water. Contact a doctor.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which user or patient is domiciled.

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