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EN Light-curing calcium hydroxide liner

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY

TO BE USED BY DENTIST ONLY

COMPOSITION: Calcium hydroxide, hydroxyapatite, urethane dimethacrylate (UDMA), barium sulphate, excipients.

INTENDED USE

TOTAL BLEND is intended for use during dental treatment as a primer before laying the composite material. Contrasting in X-ray imaging. The high filler content minimizes shrinkage and facilitates insertion of the product.

INSTRUCTIONS FOR USE

Clean and dry the treatment area prior to treatment. Then apply the bonding system according to the manufacturer's instructions. Dip the attached applicator in spirit (ethyl alcohol) and then place on the syringe. Immerse the applicator in ethanol before use, then install the applicator on the product syringe.

Apply a thin layer (0,2 mm - 2 mm) of the product to the treatment area, and use a curing lamp for 30 seconds.

If necessary, apply another layer and repeat the lamp curing also for 30 seconds.

ATTENTION: Keep dentin coverage to a minimum in order to maximise the surface area available for the restoration (do not apply the primer over the entire cavity).

Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection. Immediately after use, withdraw the syringe plunger and place the original cap on the container to avoid polymerisation of the product in the syringe.

CONSERVATION

Conserve the product in its original packaging.

Attention: Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

Use rubber dams while working with the product. Use gloves, goggles and protective clothing.

The product causes skin irritation. Causes serious eye injury. May cause an allergic skin reaction. In case of contact, rinse immediately with plenty of water and seek medical attention. If swallowed, drink plenty of water. Do not induce vomiting. Contact a doctor.

If a serious product-related event develops, report it to the manufacturer and the competent authority of the state in which the user or patient is domiciled.

STORAGE

Store in original packaging below 25°C.

Keep out of reach of children.

The expiry date is on the direct packaging.

Once container is open, the expiry date doesn't change if the container is securely closed after each use.

Protect against sunlight and keep away from heat sources.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

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