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Instructions for use and maintenance

Polymerization spectacles are designed mainly for dentists, dental technicians, medical assistants, patients for protection against the polymerization lamp light.

The set includes:

- polymerization spectacles
- case for storing spectacles

Method of use

The spectacles have adjustable temples enabling shaping the length and angle of the inclination. In order to set the spectacles firmly on your face, first you need to adjust the length and angle of the inclination using adjustable temples fitting them to individual requirements. Only replacement shields marked with CE symbol or safety sign of the equal quality can be used. The range of use for both the frame and the shield have to match. Do not use after 5 years from the date of purchase.

Method of maintenance

The spectacles should be cleaned with clean water with cleaning agent and can be disinfected with any disinfectants available on the market. The spectacles are not suitable for sterilization. Shields need to be wiped with soft cloth. Scratched, dirty, damaged or discoloured shields and/or frames need to be replaced.

The instructions for transport and storage

To be transported only in the original packaging.
Keep in a dry place in room temperature.
When disposing as waste, the country's regulations must be followed.

Warnings

Materials coming in contact with the skin can cause allergic reaction for people with sensitive skin. Damaged spectacles must be replaced, they cannot be repaired.

Restrictions for use

Polymerization spectacles are not suitable to use as a protection against fast-moving particles, laser beams, temperatures above 55 degrees Celsius, gap light interference, molten metals, materials causing infections and infectious organisms!

The manufacturer does not hold any responsibility for the consequences of inappropriate use of the product in accordance with the instructions and purpose of use.

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INFIELD Safety GmbH