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EN Material for filling and rebuilding of root canals

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COMPOSITION:

MTA+ powder: calcium oxide, oxides of: silicon, iron, aluminum, sodium, potassium, bismuth, magnesium, zirconium; calcium phosphate. *MTA+ liquid*: Ph. Eur. purified water.

INTENDED USE

The product is intended for dental treatment as a material for filling and remineralization of root canals. The compound cures in 6 to 10 minutes after mixing with water. The high content of calcium ions remineralises the tooth tissue. The silicon and calcium compounds support tissue regeneration after wall perforation and intracanal resorption. MTA+ is especially recommended

- root canal wall perforation
- intracanal resorption
- filling of the posterior root apex
- direct coating of the pulp
- pulp amputation
- treatment of teeth with interrupted root development.

MTA+ is not suitable as a standalone permanent root canal filling.

INSTRUCTIONS FOR USE

Cut off the *MTA+ liquid* pipette tip. Move the entire contents of the *MTA+ powder* glass vial with 1 to 2 drops of the *MTA+ liquid* on a mixing plate.

Mix for 30 s until the mixture has a consistency of moist sand. If the compound mix is too thick or too brittle, add one more drop of the liquid (or deionised water). Place the compound at the target site with an applicator. Exposure to moisture, including blood, will reduce the curing time. One direct coating of the *MTA+ powder* contains a single dose. Once mixed with the *MTA+ liquid* (or deionised water) the mixed dose must be used within 10 minutes. The compound cures and hardens afterwards.

CONTRAINDICATIONS

Do not use in inflammatory conditions. Acidic pH values disrupt setting of MTA+.

PRECAUTIONS

Causes serious eye damage.

Irritating to the mucous membranes of the oral cavity and skin. In case of contact, rinse with plenty of water and contact a doctor immediately. If direct contact of the product with the eyes occurs, contact a doctor immediately. Use rubber dams, disposable gloves and safety goggles when working with the product.

Discolouration might occur. Multi-stage dental treatment may involve temporary pain. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in a tightly closed original container below 25°C.

Protect from moisture. Keep away from children. The expiry date is on the direct packaging.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.
Last update: 30.11.2023/8

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