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OPAKOWANIE

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EN	Impregnated retraction cords
<b>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY</b>	
TO BE USED BY DENTIST ONLY	

**COMPOSITION:** Polyester/ Polyamide yarn conjugated. Aluminium chloride impregnated: 000.0/000/00/0 : 0,5 ± 0,3 mg/in 1/2: 1,0 ± 0,4 mg/in

INTENDED USE

Retraction cord impregnated with Aluminium Chloride Hexahydrate. Used for the retraction of gingival tissues around tooth to improve impression results and gingival edge haemostasis.

INSTRUCTIONS FOR USE

Cut the impregnated cord to the required length using the blade provided in the lid of the box. Place it tightly on the neck of the tooth and press it into the gingival sulcus using a plugger. Remove the cord when taking an impression.

CONTRAINDICATIONS

Do not use in patients allergic to aluminum chloride hexahydrate. Do not use the cord that is being cut. Check the product packaging abnormality before use. Do not use products that have expired.

PRECAUTIONS

Product for dental use only. Keep out of reach of children. This product can cause eye and skin irritation. There may be permanent gingival recession if applied for over 20 minutes. Do not put the cord deeply for patients with periodontal disease. Coagulation may occur in case of contact with alkali reactants or phosphates. Remove a cord slowly to prevent force application on the gingival tissue. The use of the inflamed area shall be considered, weighing the benefits against the risk. If a serious device-related incident occurs, it must be reported to the manufacturer and to the competent authority of the State where the user or patient is domiciled.

STORAGE

The expiry date is on the direct packaging. Close the cap immediately after use. Avoid direct sunlight and store in a dry place.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

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