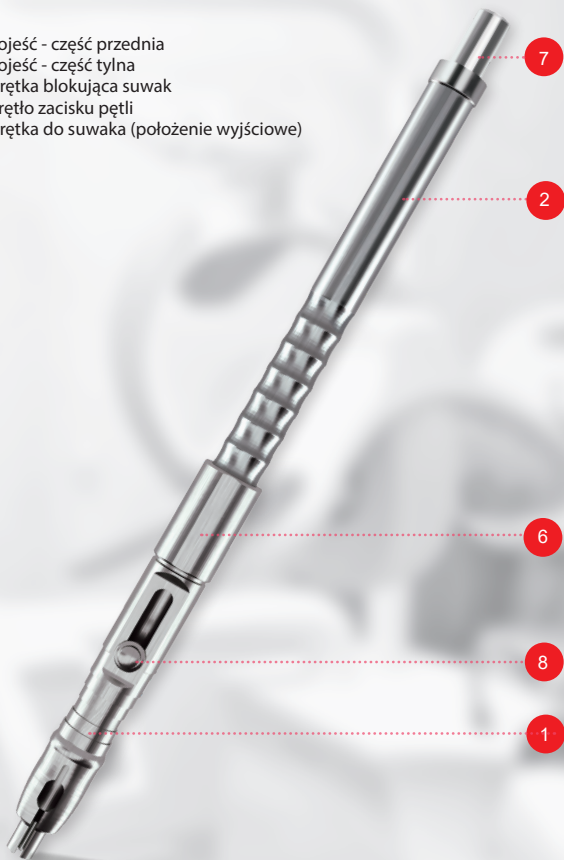
**btr** *pen* Broken  
Tool  
Remover

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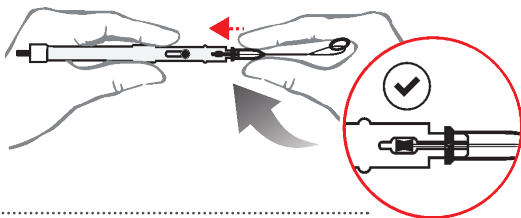
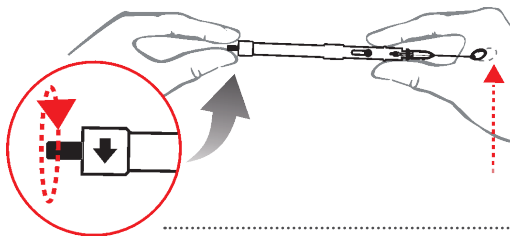
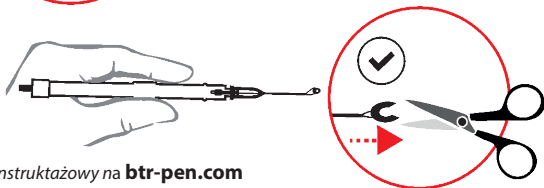
Zobacz film instruktażowy na [btr-pen.com](http://btr-pen.com)

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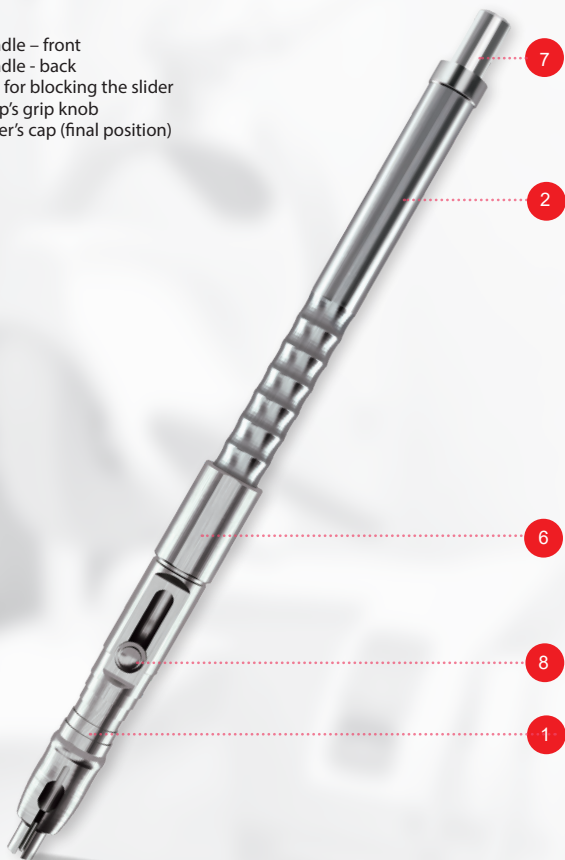
Pic. 1 **Outer construction of BTR PEN**

1. Handle – front
2. Handle - back
6. Cap for blocking the slider
7. Loop's grip knob
8. Slider's cap (final position)

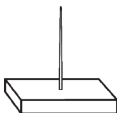
**BTR PEN with the equipment:**

BTR PEN - 1 pc  
stainless steel tool

---

BTR CALIBRATOR - 1 pc

---

BTR TIP - 5 pcs

---

BTR SPRING - 2 pcs  
soft and hard  
(third middle spring fixed on the device)

---

Silicone BTR O-RING - 3 pcs

Pic. 2 **Construction of the BTR PEN** - parts

1 Handle – front

2 Handle - back

3 O-ring

4 Main pin

5 Spring

6 Cap for blocking the slider

7 Loop's grip knob

8 Slider's cap

## Disassembling BTR PEN

1.

Unscrew the cap from the slider (8) from the front of the handle (1).

2.

From the back of the handle (2) unscrew the loop's grip knob (7).

3.

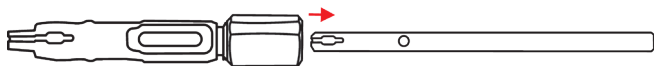
From the front of the handle (1) unscrew the back of the handle (2).

4.

Remove the spring (5) from the back of the handle (2).

5.

Remove the main pin (4) from the front of the handle (1).

6.

Unscrew the cap for blocking the slider (6) from the front of the handle (1).

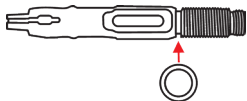
7.

Remove the o-ring (3) from the front of the handle (1).

## Assembling BTR PEN

1.

On the front of the handle (1) put the o-ring (3) in a designed place.

2.

Screw the front of the handle (1) with a cap for blocking the slider (6) (bigger diameter facing the o-ring).

3.

To the front of the handle (1) insert the main pin (4).

4.

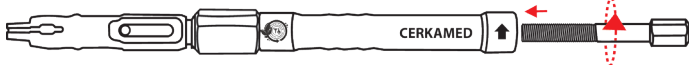
To the back of the handle (2) insert the spring (5).

5.

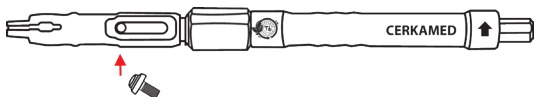
Screw the front and the back of the handle together.

6.

To the back of the handle (2) screw the loop's grip knob (7).

7.

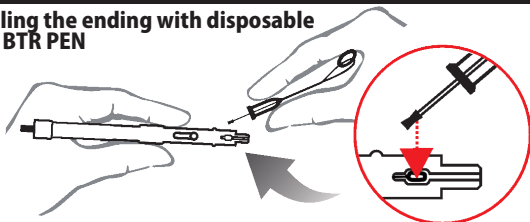
Screw the slider's cap (8) into the hole in the main pin (4) visible in the front of the handle (1).

Video of the instructions for use is available on the website: [btr-pen.com](http://btr-pen.com)

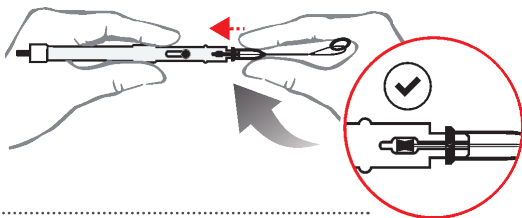
## Assembling the ending with disposable loop on BTR PEN

1.

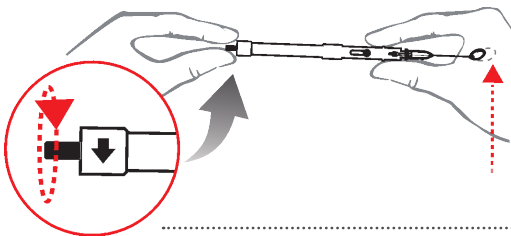
**CAUTION:** Cap for the slider must be in a final position (see pic.1). Loop's grip knob is used for regulation of the position of the cap.

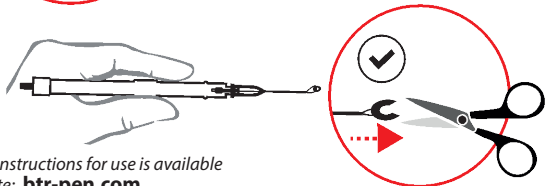
2.

3.

4.

Video of the instructions for use is available  
on the website: **[btr-pen.com](http://btr-pen.com)**

Last update: 12.01.2023/5

[btr-pen.com](http://btr-pen.com)

[cerkamed.com](http://cerkamed.com)