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EN	Liquid for root canals rinsing
PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE TO BE USED BY DENTIST ONLY	
COMPOSITION: 40% citric acid solution.	
INTENDED USE The preparation is used during endodontic treatment for rinsing root canals. It improves mechanical cleaning of the root canal. It removes smear layer from the walls of root canal exposing the dentinal tubules. It allows for precise cleaning of dentinal tubules from pulp debris and precise sealant penetration. Adapter with a plug for taking the liquid preparations into the syringe 1. Unscrew the bottle cap. 2. Screw the adapter on the bottle. 3. In order to use the bottle, remove the blue plug, screw the end of the syringe into the adapter opening, tilt the bottle with the preparation upside down and take the liquid into the syringe, while simultaneously holding down the adapter. Next, put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with a rubber plunger. 4. After each use, close the adapter opening with a blue plug. 5. For transport, remove the adapter and screw the cap on the bottle.	

INSTRUCTIONS FOR USE
Insert the product into the root canal using the syringe with a needle with a side opening while mechanically preparing the canal. After using the product, rinse the canal with physiological saline.

CONTRAINDICATIONS
Do not use the product for patients with hypersensitivity to the product's components. Do not leave the product in the canal or the tooth cavity between visits. This may result in excessive softening of dentin.

PRECAUTIONS
Take special care should be taken when the product is applied into perforations. It irritates the eyes. It may cause respiratory irritation. In case of contact, immediately rinse with plenty of water and seek medical attention. Use rubber-dam while working with the product. Use protective gloves, glasses and clothing.
Take special care when using this product to children because of the danger of excessive softening of tooth hard tissues.

DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

DE	Liquido para la limpieza de los canales radiculares
EN	Liquid for root canals rinsing
ES	Líquido para la limpieza de los canales radiculares
FR	Liquide de rinçage pour les canaux radiculaire
DE	Fluid zum Spülen von Wurzelkanälen
UK	Стоматологічна рідина для промивання корневих каналів
RU	Жидкость для полоскания корневых каналов
IT	Soluzione per lavaggi canali
RO	Lichid pentru clătirea canalelor radiculare
HU	Gyökércsatorna öblítő folyadék
BG	Течност за изплащане на кореновите канали
LT	Skystis skaknų kanalų plovimui
PT	Líquido para enxaguar canais radiculares
DA	Skyllevæske til rodkanaler
EL	Υγρό για πλύσιμο οδοντικών καναλιών
ET	Juurekanalite loputusvedelik
FI	Neste juurikanavien huuhutteluun
HR	Preparat za ispiranje korjenjskih kanala
LV	Skidums sasku kanalu skalošanai
NL	Vloestof voor het spoelen van wortelkanalen
SK	Roztok na vyplachovanie korových kanálikov
SL	Tekočina za ispiranje korenskih kanalov
SV	Vätska för sköljning av rotkanaler

STORAGE
Store in closed original packaging below at room temperature 25°C. Keep out of reach of children.
SHAKE BEFORE USE!
Expiry date is placed on the main container.
Once the container is open, the expiry date doesn't change if it the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Bottle of 200 g or 400 g of preparation, adapter with a screw cap, dispenser.
Last update: 03.01.2024/14

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