**btr** Pen Broken  
Tool  
Remover

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# BROKEN TOOL REMOVER PEN

Dental tool for removing broken canal instruments

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY  
TO BE USED BY DENTIST ONLY  
NON - STERILE PRODUCT**

The tools are made of corrosive - resistant stainless steel.

## HANDLING OF THE BTR PEN:

The stainless steel tool is intended for a reusable use, but the BTR TIP is for single use.

## HANDLING OF THE BTR TIP

The BTR TIP is delivered non – sterile. Before use, the tip should be sterilized, using standard bags according to the sterilization in autoclave. Store in a dry and clean place for storage. Observe the instructions of the autoclave manufacturer. BTR TIP is for single use.

### **STERILIZE WITHOUT THE PROTECTIVE CUP!**

Used tips disposed of responsibly or returned to the manufacturer.

## Handling dental tools made of stainless steel

Special processes have been undertaken during the manufacture of the dental instruments to ensure corrosion-resistance. In spite of this, the durability of these dental instruments depends on their proper use and correct maintenance.

## Cleaning, rinsing and washing

The packaged instruments are non – sterile. Before sterilization, these new instruments should be washed thoroughly with warm water and using detergent. Before each use these dental instruments must be sterilized.

## Sterilization

All (reusable) dental instruments must be disinfected, then washed out to remove detergent and then dried before sterilization. Ultrasonic cleaning is particularly recommended as the most effective and efficient method. Instruments should be sterilized in accordance with national requirements.

Recommended sterilisation methods:

Method	Minimal parameters:
Steam autoclave	134°C, for 5 minutes

## PRECAUTIONS

Adhere to the guidelines for cleaning and sterilization of the dental instruments. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

## STORAGE

Dental instruments should be stored in temperatures ranging between 5-30°C and relative humidity below 70%. Before use protect the instrument against contamination or damage of the material structure (cracks, bends, fractures, peeling, etc.). Failure to adhere to the above stated instructions decreases instrument durability and reduces instrument lifetime.

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