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EN	Non-latex dental dam
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY	

INTENDED USE

Non-latex dental dams are manufactured from synthetic polyisoprene polymer which has physical properties similar to those of natural rubber latex, but without the proteins that occur in natural rubber latex and that may cause allergic reactions. The dental dam is used as a protective oral barrier during endodontic procedures to reduce the risk of infection, to isolate the working area, to prevent particles or small objects making their way into the patient’s airways and to improve patient’s comfort.

INSTRUCTIONS FOR USE

- Place the dental dam on a template, then adjust.
- Use a marker to mark the point that defines a specific tooth to be isolated.
- Use a punch to cut the hole.
- Place the dental dam on a metal frame.
- Place the clamp wings in the hole.
- Using rubber dam forceps open the clamp and place it on the tooth.

CONTRAINDICATIONS

It is the responsibility of the dental practitioner to determine if the use of dental dam is appropriate for each individual patient. The dental dam should not be used in situations where the skin in the relevant area is broken, the patient has known sensitivity or other clinical considerations indicate that it should not be used.

PRECAUTIONS

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

Do not use the products if you found any holes or tearing.

STORAGE

Keep dry. Protect from sunlight. Store at a temperature between 5°C (41°F) and 26°C (79°F).

PACKAGING

The package contains 36 pieces.

VARIANTS AVAILABLE:

NON-LATEX DENTAL DAM MEDIUM

NON-LATEX DENTAL DAM HEAVY

AFTER USE, HAND OVER FOR DISPOSAL TO AN AUTHORIZED RECIPIENT. REUSE MAY CAUSE INFECTION.

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